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ENGLISH

INFORMATIVE NOTE: MODEL ZEISS SP1

The ZEISS SP1 mask was designed to protect eyes from liquid droplets and sprays. The device was not designed to protect users from impact with hard and/or sharp objects.

The ZEISS SP1 mask ensures perfect visibility and is fog resistant.

The ZEISS SP1 mask is a Category II personal protective equipment (PPE) and, as such, it is governed by (EU) Regulation 2016/425 and is in compliance with the EN 166:2001 standard.

This PPE has the CE marking and is certified by: CERTOTTICA Srl, Zona Industriale Villanova, I-32013 Longarone (BL), Italy (N.B.0530)

DEVICE MARKING:				
ZEISS	EN 166	3-4	S	CE
Identification of manufacturer	Technical standards applied	Intended use <p>3 Protection against liquid droplets</p> <p>4 Protection against large dust particles</p>	Symbol for increased robustness	CE Marking

EYEPIECE LABELING:				
ZEISS	1	S	N	CE
Identification of manufacturer	Optical class	Symbol for increased robustness	Ocular performance <p>N eyepiece fog resistance</p>	CE Marking

ANTI-FOG COATING:

- Check the device before wearing;
- Immediately replace if there are signs of damage, including marks on lenses;
- When in doubt, do not use;
- If the device is worn over prescription eyeglasses, any force of impact may be transmitted resulting in danger to the user;
- Do not modify or alter the product;
- Do not place the mask on the neck or above the head. Make sure that the device is worn safely and that it is positioned in the correct manner to protect the user’s eyes. Product performance is guaranteed if the mask is placed over the eyes;
- Damage and scratches may severely reduce the level of impact protection provided by the device;
- Materials that come in contact with the user’s skin may cause allergic reactions in users with sensitive skin;
- Ensure that the protective eyewear is worn safely and that it is correctly positioned to protect the user’s eyes;
- Adjustment the fit by adapting to the user’s face. The PPE must not accidentally move during use;
- If protection against high speed particles at extreme temperatures is required, the protective eyewear must be marked with the letter T next to the impact resistance letter. For example, FT, BT, and AT;
- If the F, B, and A symbols are not the same on the eyepiece and on the frame, the protective eyewear must be assigned the lowest level of protection;
- Eyepieces in optical class 3 are not suitable for extended use;
- PPE must not be shared between multiple users; each PPE is specifically intended for personal protection.

WARNINGS:

STORAGE:

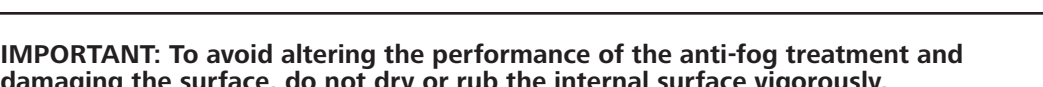
Store the device in a dry place away from direct sunlight. The protective eyewear must be transported and stored in its original packaging at a temperature of 5° ~ 40°C, with < 60% humidity. Do not leave the device near sources of heat.

Transport in its original packaging. Always store the mask in its original packaging after use (to avoid its contamination and damage).

MAINTENANCE AND CLEANING:

- External surface: We recommend using warm water and, if the mask is extra soiled, also using some liquid detergent with a neutral pH. Rinse and allow to dry. Do not clean with organic solvents or use anti-fog sprays.

- Internal surface: Do not use water, detergents, solvents, or anti-fog sprays. Gently and carefully clean with a dry cloth.

IMPORTANT: To avoid altering the performance of the anti-fog treatment and damaging the surface, do not dry or rub the internal surface vigorously. Avoid the use of any disinfectant such as alcohol / ammonia / benzalkonium chloride / chlorine / benzene / sodium hypochlorite / solvents / alkaline and acid solutions. Avoid any type of abrasive paste. If, after the cleaning procedure, the nature of the contaminants poses a safety and health risk, it is recommended to completely replace the PPE. Individuals with skin sensitivity may have allergic reactions to some of the materials within this product if they come into direct contact with their skin.

DISPOSAL OF PROTECTIVE EYEWEAR:

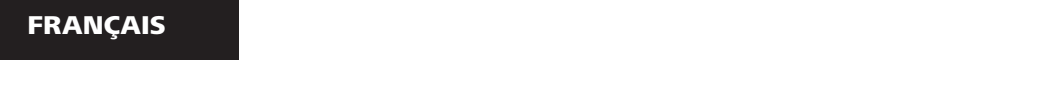
Scratched and damaged lenses have to be replaced or disposed of. Dispose of the product after 5 years.

PARTS AND ACCESSORIES:

Contact the manufacturer for additional product information.

DECLARATION OF CONFORMITY:

The Declaration of Conformity is available at the following link http://www.sunlens.zeiss.com/conformity-declaration/

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